User Manual

Alcohol Checker – Model MR1106

Available in this manual:

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This edition consolidates the original manufacturer leaflet into a single, clearly organised document. Content and specifications are reproduced from the manufacturer's original documentation.

Table of Contents

Product overview

Specifications

Breath / blood alcohol reference table

English

Français

Deutsch

Italiano

Español

Product Overview

Thank you for choosing the MR1106 exhaled-gas alcohol content detector (referred to as “MR1101” on the internal technical label). This is a high-performance personal alcohol-detection device that follows recognised national standards and has passed multiple independent testing and inspection certifications.

- Uses a high-precision, highly reliable semiconductor gas sensor.
- Accurate gas-path design, a high-precision sampling system, refined instrument-circuit design and an advanced AI algorithm ensure fast, accurate and stable measurements.
- Active blow detection makes the device convenient to use; it is compact and easy to carry.
- Fully developed in-house, from research and development through to production, backed by comprehensive after-sales support.

Applicable standards:

- GB/T 21254-2017 – National standard for expired-breath alcohol content detectors
- GB/T 19522-2010 – Blood and expiratory alcohol content threshold and test for vehicle drivers
- GB 4943.1-2022 – Audio, video, information technology and communication technology equipment – Part 1: Safety requirements

Parts:

1. Intake nozzle – align the mouthpiece with the nozzle for the puff test after installation
2. LCD display screen
3. Test-result indicator light
4. Function button / power on-off button
5. Hanging-strap hole
6. Voice-indicator sound hole
7. Air inlet

Specifications

Product name	Portable alcohol detector
Product model	MR1106 (MR1101)
Sensor	Semiconductor gas sensor
Blowing time	4 seconds
Response time	1–15 seconds
Recovery time	If the test result is >0.080 %BAC, recovery time <30 seconds
Measuring range	0–0.300 %BAC · 0–3.00 g/L · 0–1.5 mg/L
Power supply	3 x AAA 1.5 V batteries
Case material	ABS engineering plastic
Display	1.33-inch LCD display
Operation	Dual-button operation
Operating temperature	-5 °C to 45 °C
Storage temperature	-20 °C to 50 °C
Product size	120 x 59 x 20 mm
Product weight	66 g
Package contents	1x alcohol detector, 1x mouthpiece, 1x user manual

Breath / Blood Alcohol Reference Table

	1	2	3	4	5	6	7	8	9	10	11	12
Breath alcohol – mg/L	0.05	0.10	0.15	0.20	0.25	0.30	0.35	0.40	0.45	0.50	0.55	0.60
Breath alcohol – mg/100mL	10	20	30	40	50	60	70	80	90	100	110	120
Blood alcohol – %BAC	0.01	0.02	0.03	0.04	0.05	0.06	0.07	0.08	0.09	0.10	0.11	0.12
Blood alcohol – ‰	0.1	0.2	0.3	0.4	0.5	0.6	0.7	0.8	0.9	1.0	1.1	1.2
Blood alcohol – g/L	0.1	0.2	0.3	0.4	0.5	0.6	0.7	0.8	0.9	1.0	1.1	1.2

Common units by country: mg/100mL (China) · %BAC (USA) · g/L (Europe) · mg/L BrAC (Japan).

- The unit shown depends on the country setting: %BAC – United States; g/L – Europe; mg/L BrAC – Japan; mg/100mL – China. The system automatically selects the matching default unit for the selected menu language (this can be changed manually – see step 8 of the operating instructions below).
- 0.05 %BAC corresponds to 0.05 g (50 mg) of alcohol per 100 mL of blood.

English

Alcohol Checker MR1106 – User Manual

1. Operating Steps

Single-device test procedure

- 1 When using the device for the first time, prepare three 1.5 V (AAA) dry batteries, remove the battery cover, insert the batteries, then replace the battery cover.
- 2 Insert the mouthpiece into the reserved socket on the device.
- 3 Press and hold the [Power] button for 2 seconds to switch the device on.
- 4 “Warm up” (1–10 seconds): the device runs a 10-second preheat countdown – please wait for it to finish.
- 5 The screen shows “Blow” together with a voice prompt. Take a deep breath and blow steadily into the mouthpiece for 3 seconds. Once you start blowing you will hear a beep, which stops once you have blown for 3 full seconds. If you blow for less than 3 seconds, or stop halfway, the device reports that blowing was interrupted and restarts the blowing phase.
- 6 After a successful blow, the device analyses the sample and shows the result instantly. The value appears on screen and is also announced by voice.
- 7 If less than 3 minutes pass between two tests and residual alcohol is detected in the air chamber, the screen flashes immediately and the device automatically starts a high-temperature cleaning cycle. Once the screen stops flashing and displays normally again, cleaning is complete and the device is ready for the next test.
- 8 Changing the display unit: within 15 seconds of a result being shown, double-click the button to switch to the next unit (e.g. %BAC for the USA, g/L for Europe, mg/L BrAC for Japan). The chosen unit is saved automatically and becomes the default from the next start-up.
- 9 One-click access to test history: after switching the device on, briefly press the “M” button to enter history mode. Each further press of “M” shows the previous record (newest first); the screen shows “NO. XX” to indicate the data of test number XX.
- 10 Press and hold the [Power] button for 3 seconds to exit all menus and switch the device off.

2. Reading the Result

Green light – standard state

- ① Measured value below 0.3 g/L – green light on.
- ② Measured value below 0.030 %BAC – green light on.
- ③ Measured value below 0.15 mg/L – green light on.

Yellow light – “drink-driving” level

- ① Blood alcohol content ≥ 0.3 g/L and < 0.5 g/L.
- ② Blood alcohol content ≥ 0.030 %BAC and < 0.080 %BAC.

- ③ Blood alcohol content ≥ 0.15 mg/L and < 0.25 mg/L.

Yellow light, alarm sounds.

Red light – “drunk-driving” level

- ① Blood alcohol content ≥ 0.5 g/L.
 ② Blood alcohol content ≥ 0.080 %BAC.
 ③ Blood alcohol content > 0.25 mg/L.

Red light comes on and the alarm sounds rapidly.

3. Precautions for Use

- For the most accurate result, avoid smoking, eating or drinking for 20 minutes before the test. This is a general rule for all breathalysers and is not specific to this device: residual alcohol left in the mouth right after drinking is very high and will make the reading far too high.
- Before first use, or after the device has not been used for a long time, fully charge/insert fresh batteries. Open the main menu and clean the sensor in test mode before carrying out a real test.

4. Troubleshooting

Symptom	Possible cause	Solution
Fails to blow	Improper use interrupted the blow, so it did not reach 1.0 litre.	Make sure the blow is continuous and reaches at least 1 L.
No response to blowing during measurement	The mouthpiece tube is blocked by a foreign object, or saliva has entered it.	Clean the tube with a damp cotton swab.
No reading in the alcohol test	The tube is blocked; repeated high-concentration measurements; the sensor is damaged.	Check whether the tube is blocked. Open the test mode in the main menu and reset the device to zero. If the sensor is damaged, return the device to the factory for repair.
No usable reading is obtained	Sugary or alcoholic residue in the mouth; smoking, or consuming alcoholic food or drinks.	If the above has been ruled out and a reading still appears, return the device to the factory for recalibration.
Timing error	Errors in the time or date have no effect on the test result.	Open the main menu and reset the clock.
Device freezes during use	Improper operation; repeated tests at high alcohol concentration.	Restart using the reset button; open test mode in the main menu and reset the device to zero.

5. Disclaimer of Warranties

The manufacturer gives no warranty, express or implied, regarding the mental or physical condition of the user, or the ability to judge whether – or to what extent – that condition may be affected by blood alcohol. The manufacturer expressly disclaims all liability, under any legal theory, for direct, indirect, incidental, special or consequential damages of any kind. Any action taken on the basis of a reading from this device is at the sole risk of the subject or user. The manufacturer is not responsible for the consequences to persons using this device, including those later shown to have been under the influence of alcohol or whose judgement or mental or physical function was impaired by alcohol. The correlation between breath alcohol content and blood alcohol concentration depends on a number of variables, including environmental factors and the subject's state of

health. A low reading, or a short time since drinking stopped, does not mean that the subject's physical or mental performance or judgement – including reaction time – is fit to respond to an emergency.

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