

B. Braun Melsungen AG Division Hospital Care CoE Pharmaceuticals 34209 Melsungen

Contact: Ming Ming Chung

Phone: +49 5661 71-2537

To whom it may concern Email: ming-ming.chung@bbraun.com

Internet: http://www.bbraun.com

Date: September 27, 2018

Product Information Sheet - Ecolav® container for Topical Irrigation

Dear Sir or Madam,

Thank you very much for your request concerning our Topical Irrigation container Ecolav®. NaCl 0,9% B.Braun and Aqua B.Braun solution for Irrigation are sterile solutions and are compliant with the requirements and limits of the European Pharmacopoeia (Ph.Eur.)

Below you'll find the main information regarding our product:

Product NaCl 0,9% B.Braun and Aqua B.Braun Solution for Irrigation in

Ecolav®

Product Picture



Product desciption

Ecolav® is an irrigation solution container with a twist-off cap for the indications for:

- irrigation and cleansing during surgical intervention.
- irrigation of wounds and burns.
- moistening wound tamponades, cloths and dressings.
- rinsing and cleansing of bladder catheters.
- irrigation and cleansing in stoma care.
- rinsing and cleansing of instruments and equipment.
- mechanical irrigation of the eyes (the use of 30 ml container is recommended).
- nasal irrigation (the use of 30 ml container is recommended)



Page 2

Product benefits The flow of the irrigation solution can be easily addressed to the

target area by applying a slight pressure to the container.

And thanks to the container nozzle, the irrigation flow can be

precisely applied to the target area.

The wide range of volume sizes (30, 100, 250, 500 and 1.000 ml) allows to suit all requirements without incurring in unnecessary

waste of solution.

Material composition Ecolav® is an irrigation solution made of polyethylene.

The container material is Latex-free, PVC-free and DEHP-free.

Classification lla

Applicable legislation 93/42/CEE

Notifying Institute TÜV Süd Product Service GmbH

Manufacturer B.Braun Melsungen AG

Carl-Braun- Straße 1 34212 Melsungen

Germany

Product references

Reference number	Product	No. of units per box
387872	AQUA B.BRAUN 100ML	20
387873	AQUA B.BRAUN 250ML	20
387874	AQUA B.BRAUN 500ML	10
387875	AQUA B.BRAUN 1000ML	10
3570310	NACL 0,9% B.BRAUN 30ML	100
3570350	NACL 0,9% B.BRAUN 100ML	20
3570380	NACL 0,9% B.BRAUN 250ML	20
3570410	NACL 0,9% B.BRAUN 500ML	10
3570440	NACL 0,9% B.BRAUN 1000ML	10

Warning and precautions

Do not use for infusion.

Only use the solution if the seal is undamaged and the solution is

clear.

Opened containers must not be saved, quantities left over must be discarded. Re-use of single-use devices creates a potential risk of

patient or user.



Page 3

Shelf life

The shelf life of Ecolav® at ambient temperature (25°C) is 3 years and is based on stability studies which comply with internationally recognized standards and guidelines of the ICH (The International Conference on Harmonisation of Technical Requirements for Registration of Pharmaceuticals for Human Use).

Storage at elevated temperature

In specific circumstances and if required by the procedure NaCl 0,9% B.Braun and Aqua B.Braun irrigation solutions in Ecolav® may be stored in dedicated heaters for a maximum period of two weeks at a temperature of up to 40°C without negative impact on the quality.

Sterilization method

Autoclave

Method of utilization

- open the container by twisting off the cap counterclockwise
- let the solution flow by applying pressure to the container
- the container nozzle permits to precisely direct the solution flow to the target area
- the 30 ml mini-dose container allows the pinpoint application (especially indicated for the irrigation of the eyes)





Please do not hesitate to contact us if you have any further questions.

Yours sincerely,

B. Braun Melsungen AG

i. A.

Ming Ming Chung Product Manger

Irrigation solution and LVP bags

i. V.

Loredana Tamburello Product Manager

LVP - Ecoflac Plus